USER MANUAL

Shower and commode transport chair 3in1

Product code: SANI-TIM

1. Characteristics

Shower and commode wheelchair is made of powder coated aluminum frame. It features flip-up armrests for easy entry and exit for people with reduced mobility. It is equipped with a seat with wedge made of PU and removable footrests. Chair has also handle for the accompanying person and a sanitary container. The construction of the wheelchair makes it possible to slide over the toilet seat.

Wheelchair is available in two versions:


- on small wheels (5"),
- on small wheels (5") with additional set of 24" rear wheels with quick-release mechanism,

The set includes: chair left and right frame with wheels, backrest with armrests, seat with wedge, right and left footrest, sanitary container with lid, upper and lower front connecting tube, back connecting cube, screws with caps (6 pcs), locking knobs (2 pcs), Allen key.

Additionally for version 24": wheels 24" (2 pcs.) and quick-release mechanism (2 pcs), hand brake (2 pcs.)

Elements of the chair:

- A. Handle
- B. Backrest
- C. Armrest
- D. Seat
- E. Sanitary container
- F. Footrest
- G. Armrests flip-up lock
- H. Parking brake
- I. 5" wheel with brake
- J. Rear 24" wheel with handrims
- K. Quick release mechanism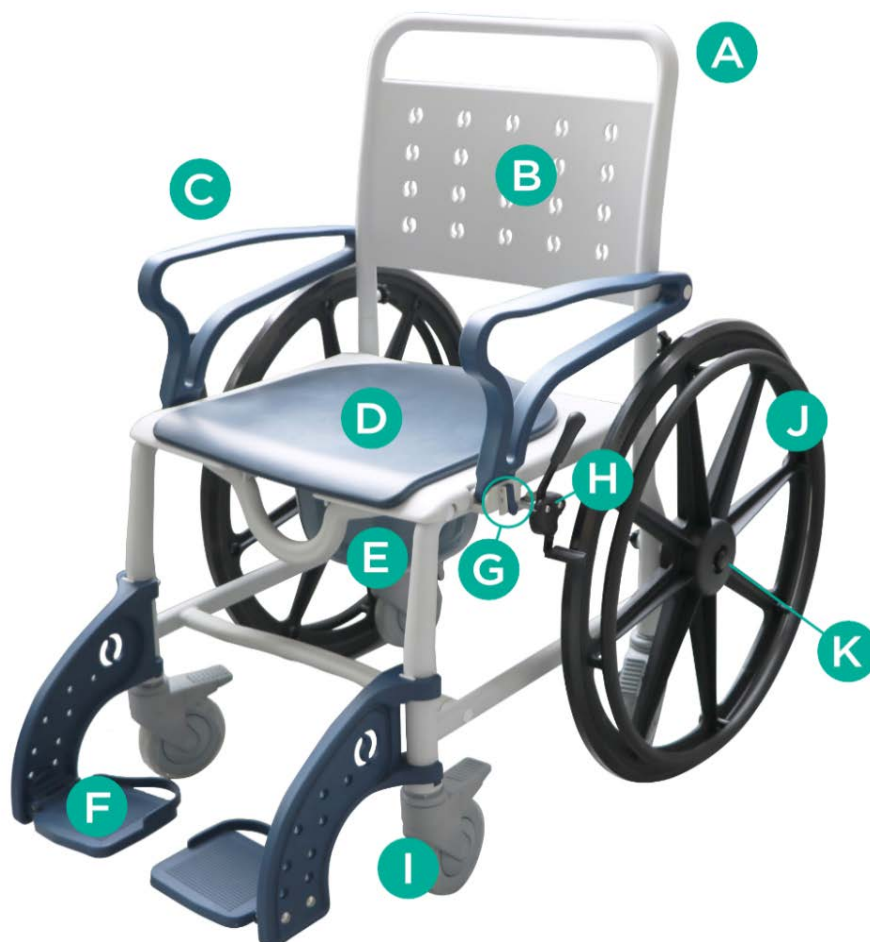


2. Application


Shower and commode wheelchairs are designed for individuals who have difficulty moving independently, as well as for patients with dysfunctions or reduced mobility. The wheelchair is intended for indoor use only. It serves both as a shower and a commode chair, thanks to a moisture-resistant aluminium frame and a sanitary container located beneath the seat. It facilitates daily hygiene routines. The wheelchair is especially recommended for people with mobility impairments caused by conditions such as paralysis, obesity or extreme underweight, limb deformities or degenerative diseases, joint degeneration or injuries, limb injuries or amputations, and cardiovascular or circulatory disorders.

3. Assembly

1. To prevent wheelchair frame from moving during assembly, lock the brakes by pressing brake levers down.
2. Connect the right and left frame with tubes using screws and Allen key (picture 1). Put the caps on the mounting openings afterwards (picture 2).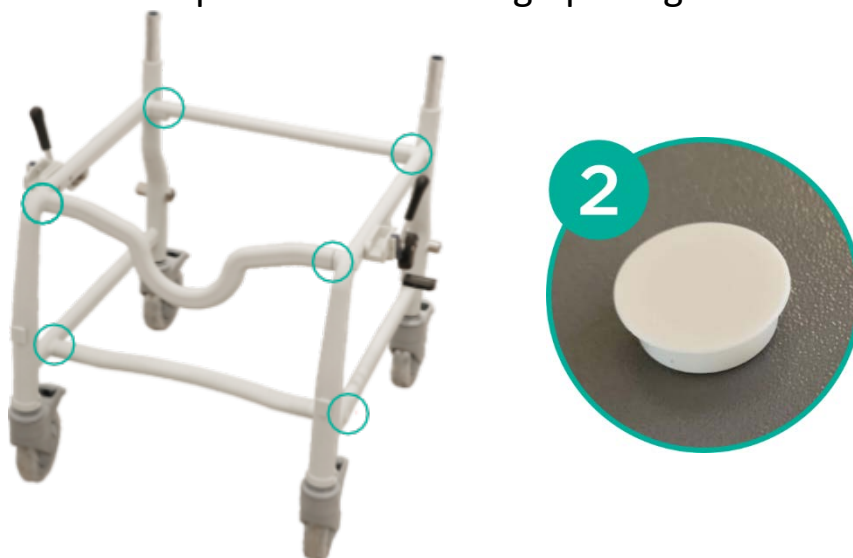

3. Install the seat on the top of the frame. Slide sanitary container under the seat from the back.
4. **Mounting of rear wheels 24"**: Insert the quick release mechanism (K) into the rear wheel opening. Make sure that the middle ring inside the opening is in center position and is not covering opening (picture 3 and 4). Put the mechanism inside the opening, pressing simultaneously black cap on it (picture 5). After mechanism is on the wheel, put it inside the opening on the rear frame of the chair, pressing black cap again. Correctly mounted mechanism should go through the opening on the other side of the frame tube (picture 6). Repeat on the other side.

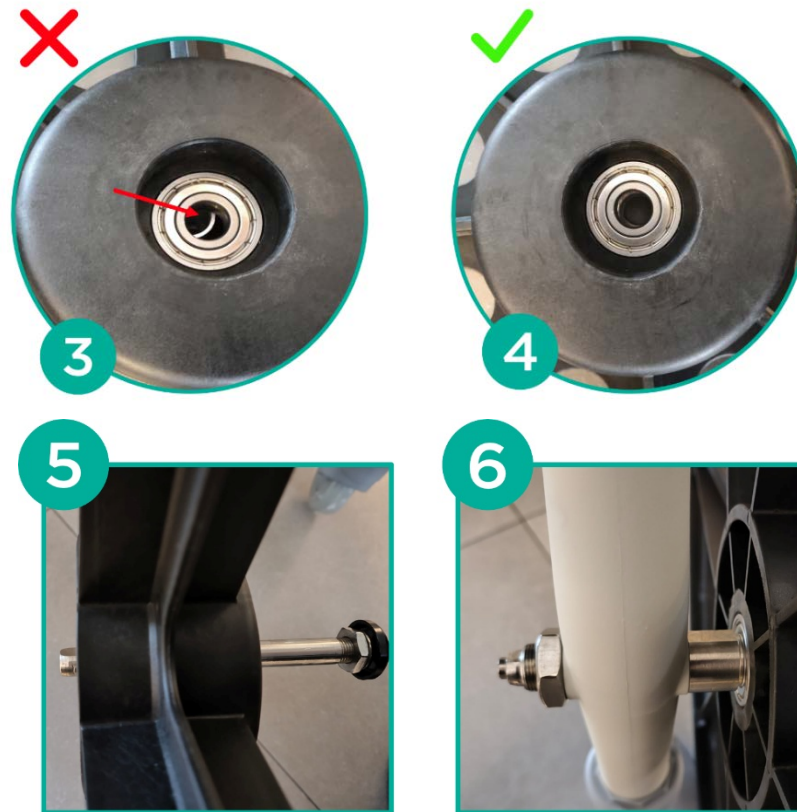5. Put backrest tubes onto chair frame and secure them with locking knobs. Press the armrests lock and lower the armrests till they're secured in place.
6. Fix the footrest on the front frame elements (picture 2).

WHEELCHAIR HANDLING

- Check the condition of the parking brakes, construction and castors. Check the proper assembly of the components and their adjustment.
- Before using, make sure that the chair's width matches the user, as the chair is not equipped with width adjustment.
- The chair is designed to be used for short distances and inside only. The chair doesn't have a shock absorber, so long-term use is not recommended.

NOTE! Do not use the chair if any defects or deficiencies are noticed.

FLIP-UP FUNCTION OF ARMRESTS

- Press the lock (G) located under the armrest and lift the armrest upwards with a gentle movement.
- **Do not** use the armrest as a support while standing up. If help is required – ask for assistance.

REAR 24" WHEELS QUICK RELEASE FUNCTION

- Rear 24" wheels can be easily removed thanks to the quick release function, allowing to use it as shower or toilet chair. To remove rear wheels, press black cap of quick release mechanism (K) on the wheel and take it out.
- To mount it back, press black cap of the mechanism and put it back into opening on the rear frame.

BRAKES

- To lock the front castors, press the pedals with your foot. To unlock the wheel, lift up the pedal.
- To lock the rear wheel, pull the parking brake lever onwards. To unlock the wheel, pull the brake lever in the opposite direction.

GETTING ONTO THE CHAIR

- It is essential to lock the brakes while getting in or out of the chair, changing the user's position or getting off.
- While changing place, the chair should be located as close as possible to the place of transfer. The chair should touch the bed or other seat being used. If the user cannot change his/her seat independently, transfer the user to the chair.
- The user should sit as deep as possible in the middle of the seat in order to avoid chair instability.
- **Do not step** on footrest while getting onto or out of the chair. This may cause the damage.

4. Technical data

MODEL	SANI-TIM 5"	SANI-TIM 24"
Overall width	58 cm	68,5 cm
Inner width	50 cm	
Overall depth	76 cm	104 cm
Seat height from the ground	50 cm	52 cm

Overall height	98 cm	100 cm
Seat dimensions	50 x 45 cm	
Weight	14,50 kg	22 kg
Front wheels	5" swivel	
Rear wheels	5" swivel	5" swivel + 24" with quick-release mechanism and handrims
Max. capacity	120	

5. Notes

Timago International Group does not bear responsibility for improper use of the product and failure to comply with safety regulations.

6. Cleaning and maintenance

Carry out the maintenance of the chair components regularly:

- The seat can be cleaned with a damp cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge.
- Plastic elements should be cleaned with damp cloth with the use of appropriate detergents for plastic elements. In such case follow the instructions given by the producer and safety measures.
- Empty and disinfect the toilet bucket and toilet seat after each use. Use cleaning products according to producer's recommendations.

7. Storage and transport conditions

The product should not be exposed to direct sunlight.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that, for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

10. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745)		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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